

Mobile Track Installation Instructions

1. With a pencil, measure and mark the centre of both pieces of slotted track.
2. Insert pan head screws into pre drilled holes at the ends of the slotted tracks. Just far enough so that the screw pokes out the back of the track.
NOTE: Use 1½” pan head screws MAXIMUM in the slotted track to be fixed on the bedscreen track side. 2” pan head screw can be used to secure the slotted track to the wall.
3. Assemble and insert trolley assembly onto the slotted track.
4. Mark the location where the centre of the slotted track will sit on both the track and back wall.
5. Hold the slotted track in position against the bedscreen track and with a hammer gently tap the screws to mark the positions for the screws on the bedscreen track.
6. Using a 1/8” drill bit drill through the bedscreen track where the slotted track is to be fixed. Screw slotted track to bedscreen track.
7. Measure and mark the height of the slotted track on the wall. Hold the track in position, ensuring it is level and with a hammer once again gently tap the screws to mark the position of the screws. The wall type will determine the fixing required.
8. Screw the slotted track to the bedscreen track.
9. Measure the exact length of bedscreen track required between the two pieces of slotted track using a tape measure. While measuring hold the trolley assemblies upright while measuring.
NOTE: It is important that the bedscreen track length is measured and cut accurately, if it is too short or too long it will cause the trolley assembly not to run correctly.
10. Once cut, insert the bedscreen track into the trolley assembly and mark the track through the holes on the side of the carrier. Remove from trolley assemblies and using a 6mm drill bit drill through both sides of the bedscreen track.
11. Reinsert the bedscreen track into the trolley assemblies and secure with a bolt and nylock nut.

